

REPLACEMENT OF CWP CONTACTORS CON 1, CON 2, CON 3 OR AUX CONTACT.

The 100-C series A/B contactors used in CWP have been replaced with the 100-E series and the two series are not interchangeable. If replacement is needed the following parts should be used.

1. If Con 1 or Con 2 need be replaced use W2T983072 (CONTCTR,100,12A,3 PH,120V,A/B).Includes contactor only.
2. If Con 3 needs to be replaced, you must also replace the Aux contact. Use W3T585982 (KIT,CONTCTR/AUX REPLT,CWP,AB). Includes contactor and aux contact.
3. If Con 3 aux contact needs to be replaced, you must also replace Con3. Use W3T585982 (KIT,CONTCTR/AUX REPLT,CWP,AB).Includes contactor and aux contact.

OLD 100-C SERIES



CONTACTOR



AUX CONTACT

NEW 100-E SERIES



CONTACTOR



AUX CONTACT

4. Attach new 100-E series aux contact to contactor (as needed).