

SDS MAINTENANCE CHECKLIST

Month	1	2	3	4	5	6	7	8	9	10	11	12	24
** Monthly Requirements **													
AAMI Bacterial Monitoring													
Bicarb/Acid Loop Leak Inspection													
SDS Mix Frame Leak Inspection													
Bicarb/Acid Head Tank Leak Inspection													
Check Bicarb Mix Tank Vent Filter													
SDS Mix Frame Wash Down													
** Quarterly Requirements **													
Bicarb System Decalcification													
Check Overflow Line for Obstructions													
Check Mix Tank Seal													
Check Inlet Water Filter													
Check Remote Monitor (if applicable)													
Perform Check Valve Test (Tech Note 297)													
** Semi-Annual **													
Replace SDS Inlet Water Filter													
Check All indicators Lights and Alarms													
Replace Bicarb Wand Quick Disconnect													
** Annual **													
Change UV Lamp													
Tighten All Terminals in Controller													
Verify Float Switch Operation													
Replace Mix Pump Seals and O-rings													
**Bi-Annual **													
Rebuild Acid/Bicarb Delivery Pumps													