

**Minnicare® HD Disinfectant and Minncare HD Test Strip Expiration Dates**

This Tech Note discusses expiration once Minncare HD Disinfectant and Minncare HD Test Strip containers have been opened.

**Minnicare HD Disinfectant**

The expiration date of Minncare HD Disinfectant remains unchanged from the expiration date printed on the label, as long as they are stored in the original container, the container is closed using the original vented container cap, the container is stored in a cool, dry place (below 75°F), and is not exposed to direct sunlight.

Product that has been dispensed from the original container and is not used should not be returned to the container, but should be discarded in accordance with all federal, state and local regulations.

**Minnicare HD Indicator and Residual Test Strips**

The expiration date listed on the Minncare HD Indicator (i.e., 1%), and Minncare HD Residual Test Strip vials list the month and year of expiration, this expiration date is an “inclusive” date.

If the test strips are properly stored and handled, they are valid and can be used through the last day of the month listed on the container, even after opening.

Minnicare is a registered trademark of Medivators, used under license.