

WRO 300/300 H RO MACHINE(S) NEW 4.3 CPU AND DISPLAY

The CPU, Display (User Interface), and Cable are now obsolete for the WRO 300 RO machines with software version 4.2 and lower due to components used during manufacturing of the boards being discontinued. There is virtually no difference between 4.2 and 4.3 software. See references to help determine what components are needed to order. There is no stock remaining of discontinued parts.

This affects machines built prior to Mar Cor Purification, now Evoqua Water Technologies taking over manufacturing of the WRO 300 RO machines in late 2010. All units built after that date have the latest version boards and software.

Highlights

- The new 4.3 CPU, Display, and CPU Display Cable are not compatible with previous versions, meaning that if one is needed to be changed, all must be changed.
- The new 4.3 software is not compatible with older components.
- The fan control board on older versions is integrated into the new CPU board; the existing fan board is not needed if CPU board is upgraded.
- The new CPU board uses a USB cable to communicate with a PC. A new cable is available.

Ordering Information

	Original Discontinued Part Number	WRO 300	WRO 300 H
4.2 CPU Board	WTW502001003	W3T577658*	W3T577659*
4.3 CPU Board	WTW502001004	W3T577658	W3T577659
Display	WTW506015001	W3T577806*	
Display/CPU Cable	WTW505001001	W3T572512*	
4.3 USB Logging Cable	N/A	W3T572607	
4.2 & Older Logging Cable	N/A	W3T572608	

Reference Documents

Contact Technical Support for the following:

W3T575396	WRO 300 4.3 Software Release and Presets
W3T575395	WRO 300 H 4.3 Software Release and Presets

* To use, CPU board, display and cable must be upgraded.