

### WRO 300/300 H & CWP WATER SYSTEMS PHOENIX CONNECTIONS


*This is an update to what was previously WATER, SP, 10058-7/2007.* The following information is intended for customers who use both a CWP and WRO 300/300 H water systems and want a uniform connector for the Gambro® Phoenix dialysis machines. The following parts and instructions can be used to modify the stainless steel (SS) valve body used on the WRO water system product loop to a CWP water system SS connector. This will allow the customer to easily move a dialysis machine from the CWP water system chase box to the WRO water system without the need of any other adapter. Descriptions and pictures of the dialysis machine inlet water line adaptors currently in use are included.

#### Required Parts

Part Number	Description
W2T913084	Stainless Steel 1/4" Nipple
W2T913265	Chase Box Valve Connector
W3T572513	Stainless Steel Y Connector

#### Instructions

1. Remove the valve stem from the WRO water system connector by holding the body with an adjustable wrench and turn the bottom of the stem using a narrow wrench or vise grips.
2. Apply 4-5 wraps of PTFE tape to one end of the SS nipple (W2T913084) and insert it into the WRO water system connector and tighten by hand.

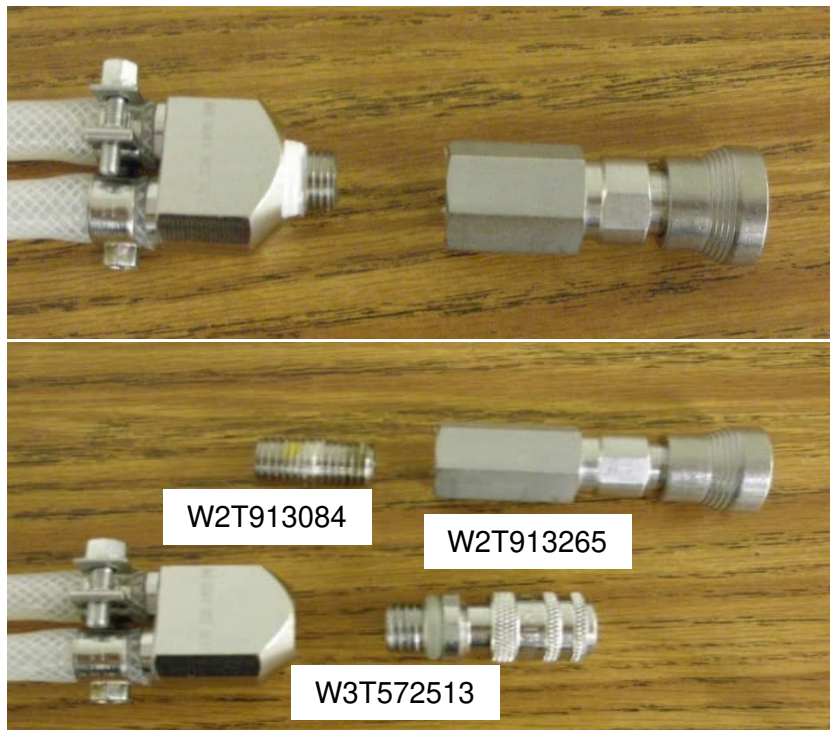

3. Apply 4-5 wraps of PTFE tape to the other end of the SS nipple and connect it to the CWP water system connector (W2T913265). Tighten the parts together using two wrenches. Both ends of the nipple connection will tighten as you turn the connector.




### Other Connectors for use with the WRO and CWP Water Systems


Part Number	Description
W3T572653	Hose Adapter to CWP
W3T577804	Hose Adaptor to WRO
W2T913665	PEX Tubing Connector to CWP
W2T918662	PEX Tubing Kit
W3T579340	Quick Connect Phoenix to WRO



W3T572653

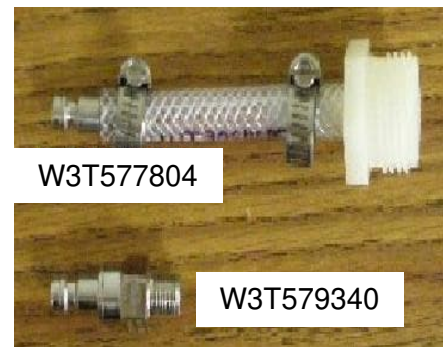

W3T577804

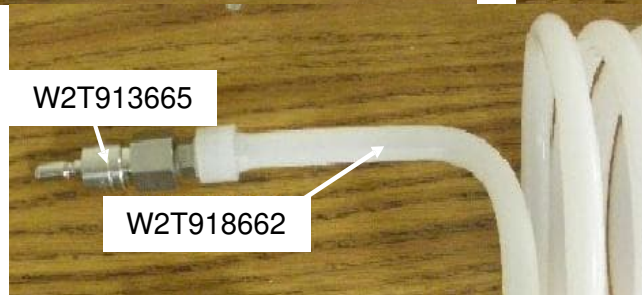

W2T913665

W2T918662

W3T579340