

MILLENIUM HX® WATER PURIFICATION SYSTEM PLC AND I/O MODULE CONNECTIONS

To mistake proof the power connections to the PLC and I/O module, connections J4 and J5 are being combined into a single connector. Replacement PLCs and I/O modules (replacement PN W3T577670) can still be used with existing machines without any wiring modifications by plugging both J4 and J5 into the new single connection (see Figure 1).

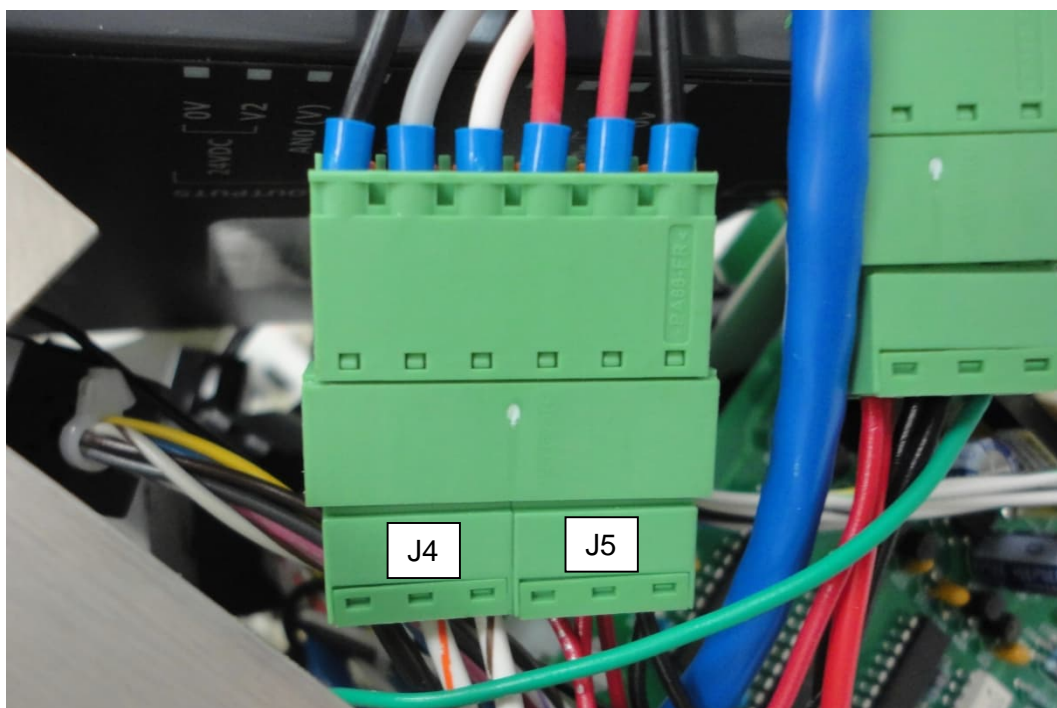


Figure 1. J4 and J5 Connection

Millenium HX is a trademark of Evoqua Water Technologies LLC, its subsidiaries or affiliates in some countries.