


CWP RO WATER SYSTEM INDICATION PANEL (LIGHT STACK) IMPROVEMENTS


The indication panel design for the CWP RO Water System has evolved over time. This document explains the changes to the indication panel, replacement components available for servicing and ordering information when upgrading to the new design. The indication panel is used to visually display the CWP water system status and alarms by color according to the diagram below:

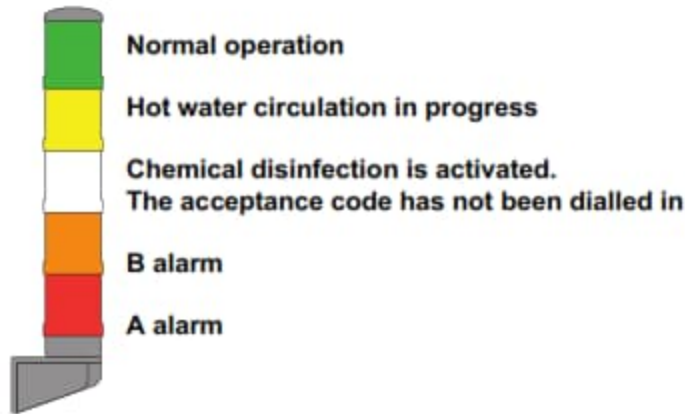

An indication panel should be mounted on/near each CWP water system unit (part of local alarm) and at critical locations in the dialysis facility such as the patient treatment area and/or the nurses station (part of the remote alarm).

Current Indication Panel (P/N W2T914336)

The current indication panel (P/N W2T914336) has been supplied standard with the CWP since August 14, 2014. This light stack is non-repairable; a failure of any of the lamps requires replacement of the unit. The images below show the indication panel on the CWP water system and the remote alarm.

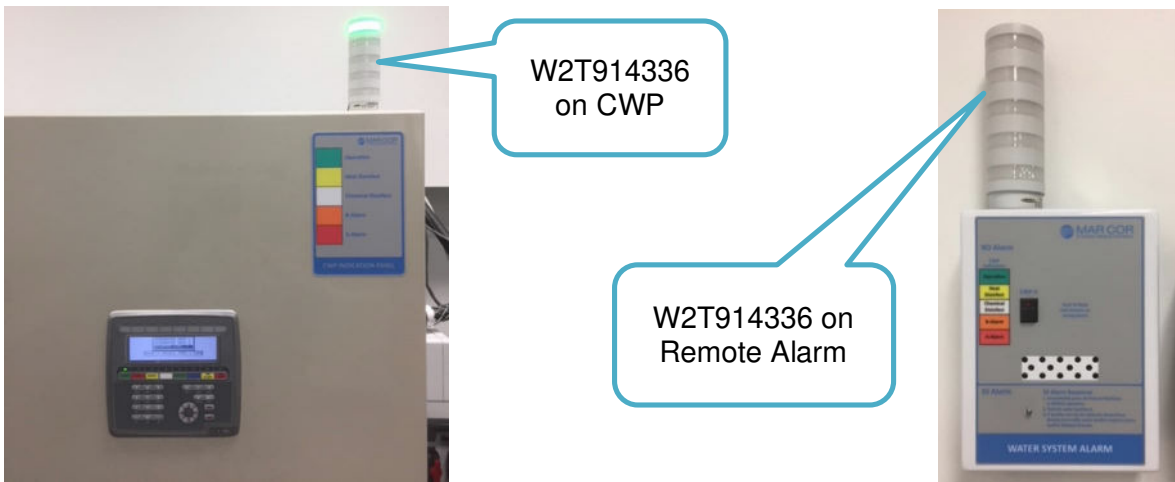

All orderable parts related to the current CWP water system indication panel are listed in the table below:

| CURRENT INDICATION PANEL ORDERING INFORMATION | |
|---|--|
| P/N | Description |
| W2T914336 | LIGHT,STACK,8-PIN QD,24V AC/DC,G/Y/W/A/R |
| W2T923148 | CORD,8-PIN EURO,STR,2FT CABLE |
| W2T913694 | CORD,8-PIN EURO,STR,6FT CABLE |
| W2T913444 | BRACKET,RIGHT ANGLE,LIGHT STACK,BAN |
| W3T577691 | KIT,RO&DI REMOTE,CWP H |
| W3T578224 | KIT,RO REMOTE ADDER,CWP S |
| W2T913255 | REMOTE ALARM CABLE, 250 FT |

Upgrading To Current Design

Order W2T914336 and W2T913694 if upgrading the indication panel on the CWP water system (local mounting) to the current design. If desired, mounting the indication panel to a vertical surface requires use of bracket W2T913444.

If upgrading the CWP water system remote alarm to the current design; order kit W3T577691 and cable W2T913255. If utilizing a CWP water system supplemental (dual CWP H/S system); kit W3T578224 and a second cable W2T913225 must also be ordered.

Previous Indication Panel (P/N W3T575730)

The previous indication panel (P/N W3T575730) was supplied standard with the CWP until August 14, 2014. Individual LED modules can be replaced on this light stack if they fail. Refer to the table below for ordering replacement parts:



| PREVIOUS INDICATION PANEL ORDERING INFORMATION | |
|--|--|
| P/N | Description |
| W3T577998 | KIT,CWP H INDICATION PANEL,WALL |
| W2T923563 | ASSY,STACKLIGHT,24V AC/DC,WM,G/Y/W/A/R |
| No Longer Available | BASE/CAP,STACKLIGHT,24V AC/DC-120/240VAC |
| W2T914324 | LED MODULE,STACKLIGHT,48MM,24V AC/DC,GRN |
| W2T923564 | LED MODULE,STACKLIGHT,48MM,24V AC/DC,YEL |
| No Longer Available | LED MODULE,STACKLIGHT,48MM,24V AC/DC,WHT |
| No Longer Available | LED MODULE,STACKLIGHT,48MM,24V AC/DC,AMB |
| No Longer Available | LED MODULE,STACKLIGHT,48MM,24V AC/DC,RED |
| W2T923563 | BRACKET,STACKLIGHT,48MM,WALL MOUNT |

Original Indication Panel (P/N WT2670108)

The original CWP water system indication panel (P/N WT2670108) and its replacement components are no longer supported by the manufacturer, refer to the images below for details.

Panel WT2670108



WT2670122



WT2670123



WT2670124



WT2670125



WT2670126

A new indication panel (light stack) must be ordered in the event that WT2670108 (or its components) fails. Refer to **Upgrading To Current Design** for details.