

WRO 300/300 H PORTABLE WATER SYSTEM EXTERIOR CLEANING

This Tech Note addresses acceptable exterior cleaning products that can be safely used on the WRO 300 and WRO 300 H Reverse Osmosis (RO) machines.

For exterior cleaning purposes, wipe the outside of the WRO 300/300 H RO with a clean cloth moistened with a recommended solution or with a designated disposable wipe. The following chart identifies a variety of cleaning solutions and wipes safe for these portable RO machines.

Exterior Cleaning	<ul style="list-style-type: none">• Ethanol (70%)• Isopropanol (60%)• Soapy water• Bleach Solution (1:100 dilution)
--------------------------	--------------------------------------------------------------------------------------------------------------------------------------------------------------

On an as needed basis, the outside of the machine may be cleaned with one of these freshly prepared solutions. Use surface cleaning agents sparingly to avoid excess cleaner from entering the interior of the machine. When completed, be sure to wipe off cleaning solution with a water-dampened cloth.

NOTE: Do not use iodine-based or tenside-containing disinfectants as these may crack or discolor the plastic materials.

Contact us at 888-595-0665 if you have any questions or require any additional information.