

Millenium HX® Chemical Disinfection Frequency Change Notification

Chemical disinfection utilizing MINNCARE® HD will no longer be required quarterly and will be recommended to be performed on an “as needed” basis. Heat disinfection is the preferred method to maintain bacterial control.

This Tech Note establishes this directive and is intended to serve as notice of this recommendation on all Millenium HX RO’s. While we process updating all necessary operating manuals and other forms of literature, this Tech Note is written to supersede any current documentation that recommends quarterly chemical disinfection of the Millenium HX RO.

MILLENIMUM HX® is a registered trademark of Mar Cor Purification (a Cantel Medical Company)
MINNCARE® is a registered trademark of Medivators Inc. (a Cantel Medical Company)