

WRO 300/300 H New 4.3 CPU and Display

The CPU, Display (User Interface) and Cable are now obsolete for the WRO 300 with software version 4.2 and lower due to components used during manufacturing of the boards being discontinued. There is virtually no difference between 4.2 and 4.3 software. See references to help determine what components are needed to order. There is no stock remaining of discontinued parts.

This affects machines built prior to Mar Cor Purification, Inc. taking over manufacturing of the WRO's in late 2010. All units built by Mar Cor have the latest version boards and software.

Highlights

- The new 4.3 CPU, Display, and CPU-Display Cable are not compatible with previous versions, meaning that if one is needed to be changed, all must be changed.
- The new 4.3 software is not compatible with older components.
- The fan control board on older versions is integrated into the new CPU board; the existing fan board is not needed if CPU board is upgraded.
- The new CPU board uses a USB cable to communicate with a PC. A new cable is available. The new cable is backwards compatible with all older CPU's.

Ordering Information

	Discontinued Part	WRO 300	WRO 300 H
4.2 CPU Board	WTW502001003	3028192	3028193
4.3 CPU Board	WTW502001004	3028192	3028193
Display	WTW506015001	WTW506015002	
Display/CPU Cable	WTW505001001	WTW505001002	
USB Logging Cable	N/A	WTW300236001	

Reference Documents

Contact Mar Cor Technical Support or your local Sales representative for the following.

WATER,10099-8/2010	WRO 300 4.3 Software Release and Presets (Includes USB cable installation instructions)
TAA.TU.10100	WRO 300 H 4.3 Software Release and Presets (Includes USB cable installation instructions)